

ARROWPACK

A passion for healthcare innovations

Push Indicator Dye Test

ARROWPACK Push Indicator Dye Test is a new generation of Ink Test for routine testing of your different sealing devices

ARROWPACK Push Indicator Dye Test is specially developed to check the integrity of your seals, produced with your different heat sealer. (for Paper and Tyvek packaging)

With this ink test you can easily, safe and quick, perform a ink test according to the ISO 11607-1 as written in appendix B.

With this first ARROWPACK Push Indicator Dye Test you can safely do your own Quality Dye test without damaging the seal (no using of pipettes or other strange objects for filling).

The ARROWPACK Push Indicator Dye Test is packed (30 pieces) in a special pack for easy applications, the aluminum package will protect the test ink of light influence.

An other great advantages of ARROWPACK Push Indicator Dye Test is that you can prepare and seal your test pouch in advance, without the risk of leakage or migration of blue methylene on value objects in your work area.



How to use Push Indicator Dye Test

1. Step one



Put the test in The pouch or sleeve

2. Step two



Seal the pouch or sleeve

3. Step three



Press the test in the direction of arrow

4. Step four



4. Observe 5 sec per seal



Observe 5 seconds per seal. Discard the tested pouch after 20 seconds!

Its 's recommended to document the results on the Push Indicator Dye Test sheet, which is supplied by ARROWPACK.

	Cat. No.	Qty/box
Push Indicator Dye Test for sterilization Paper based packaging	44100	30
Push Indicator Dye Test for plasma sterilization Tyvek based packaging	44110	30