

CLARAVUE


CLARAVUE is the best-in-class, hygienic solution for extremely precise, fully radiolucent patient monitoring.

CLARAVUE includes pre-wired and pre-positioned disposable electrodes and patented reusable bioactive trunk cables.

For the past ten years, CLARAVUE has been the proven choice for superior trace quality and infection prevention in prominent European hospitals. We are pleased to be bringing CLARAVUE to you as the product becomes the standard of care in the European market for Cath & EP labs.




INSTRUCTIONS FOR USE

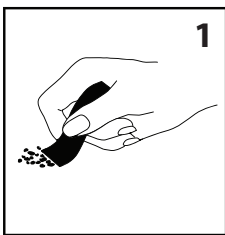

1

Standard protocol for skin prep applies.

● Prep

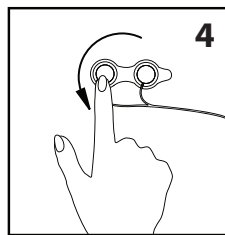


4

Adhere the Ra and Rf electrode positions to the patient. Continue to apply set A across the patient's chest to the La position and down to the Lf position. Repeat with set B, applying the C positions from left to right. CLARAVUE is repositionable, allowing the freedom to rearrange on the patient as necessary.

Note: Gently pull the wire from the ribbon cable to expand the overall size of the set(s).


● Apply

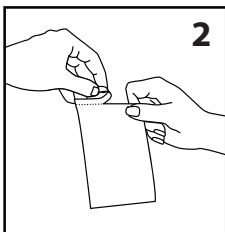

2

Open the single CLARAVUE patient kit. Remove inner bag. Unwind sets from the holder card.


● Open

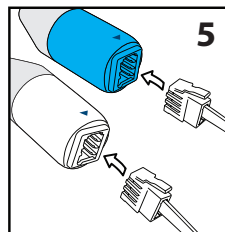

5

While holding the male end of the adapter (clear clip) attached to set A, align with the blue, triangular indicator. Press firmly until lever snaps. Repeat with the blue clip attached to set B.

● Insert

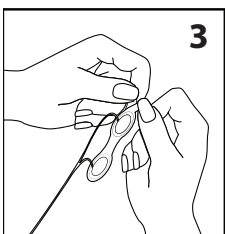

3

Gently remove electrode sets from release liner by peeling away from thumb tab.

● Peel



● < 5 days



Ensure a good signal is achieved on the monitor. If a standard three lead trace is necessary, unplug set B from the adapter cable and remove the corresponding V positions from the patient. CLARAVUE's various lead configurations enables total hospital standardization.

For More Information:

Customer Service
ukinfo@vermed.com
+44 1803 860100

CLARAVUE is smart and reliable, featuring various lead configurations to suit your specific needs, and a wide array of compatible trunk cables for all popular OEM monitors. The trunk cable has a patented connector which reduces your total leadwire connections.

CLARAVUE RADIOLUCENT DISPOSABLE PRE-WIRED ELECTRODES

Product #	Description	Per Kit	Kits/Box	Kits/Case	Solid Gel	Duration of Use
4009832C	Pediatric Patient Kit - 3 electrodes	3	40	480		 5 Days
4009837C	Adult Patient Kit - 4 electrodes	4	40	480		
4009838C	Adult Patient Kit - 5 electrodes	5	30	360		
4009839C	Adult Patient Kit - 10 electrodes	10	25	300		
4009840C	Adult Patient Kit - 6 electrodes - C-Series	6	30	360		
4009841C	Adult Patient Kit - 5 electrodes - C-Series	5	30	360		


PACKAGING

Electrodes are packaged in a single-use pouch with an easy-to-open tear

Single-Use Kit



ELECTRODE PLACEMENT

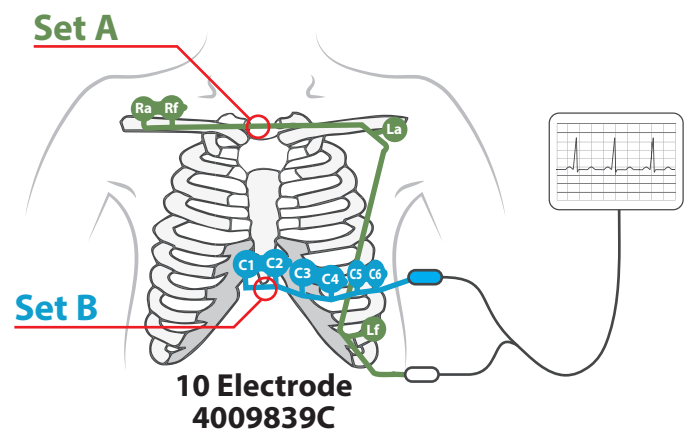

CABLE CONNECTIONS

